CERTIFIKACE

**GALAXY**

ver 1.1

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**EN**

Scheme & specs .....	2
Important .....	3
How to use .....	4
Messages .....	4
Do not do this .....	5
Calibration .....	5
Service, warranty, calibration .....	6

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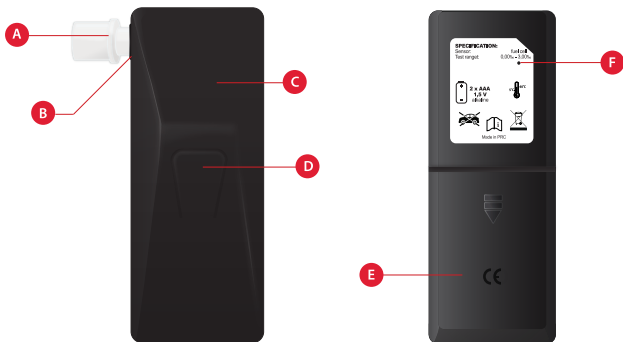
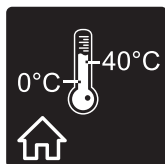
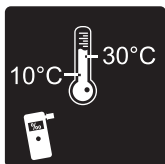
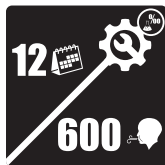
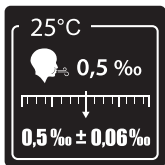
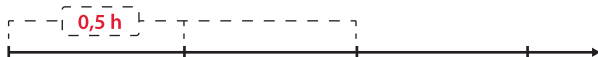
zapnite prístroj

vykonajte 3 merania


**[EN] Perform 3 measurements one by one and accept the highest one as the appropriate.**

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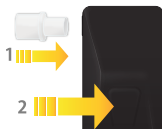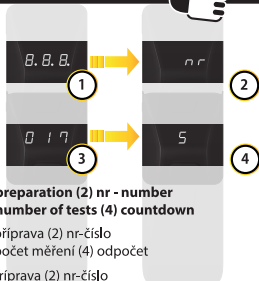
EN **perform a long, strong exhale**

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4

EN **after a „beep” sound the device will analyze  
your sample and show the result.**

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## DO NOT DO THIS

**EN** Do not smoke, do not eat, do not drink 30 minutes before the measurement.

**Do not blow into the breathalyzer immediately after drinking alcohol.**

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## CALIBRATION

**This device should be calibrated:  
every 12 months or every 600 measurements.**

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**The sensor of every breathalyzer, even if not used, is subject to gradual wear, which affects the performance of the breathalyzer. The reliability of the measurement is obtained thanks to periodic calibration of the device.**

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**Results significantly differing from the reality do not indicate the damage of the breathalyzer, but the need for its calibration.**

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## SERVICE, WARRANTY, CALIBRATION

### READ BEFORE SENDING THE DEVICE TO SERVICE:

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**EN** The alcohol tester you are using is a high quality screening device (not evidence), which - as every other technical device - needs a bit of knowledge about it's specimen. Before you decide to send your device to service read user's manual carefully. There is a high chance that you'll find the solution for your problem here.

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To claim a warranty / calibration please scan the QR code or type this adress directly

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Tel. +420774411547

[info@warscorp.com](mailto:info@warscorp.com)

**Work and Road Safety  
Corporation s.r.o.**

Kolbenova 610/26  
19000 Praha, Vysočany  
IČ: 05562686  
DIČ: CZ05562686

**Tel. +420774411547  
info@warscorp.com**